


INSTRUCTIONS FOR USE

- Maximum weight limit is **600 lbs.** per ramp set (300 lbs. per rail).
- WARNING:** Do NOT walk on rails.
- Read and follow all instructions before use.
- Do not use if ramp is damaged or unstable.
- For use with some manual and power wheelchairs.
- Remove all parts from box and verify that you have received two ramp rails, each consisting of one top, middle, and bottom ramp section; instructions; warranty info; nylon cord; clevis pins; and storage tote.
- UNPACKING YOUR RAMP RAILS:** Your ramp is packaged and telescoped inside of itself to protect it during shipping. To prepare the ramp for use, minor assembly is required. You must extend the top section by pushing in the catch button and pulling the top section out until the catch button pops into upper hole in top section (see DIAGRAM ITEM 1) and the turned-down tab (top flap) is exposed. Repeat on second rail. **IMPORTANT:** Catch buttons on the ramp are intended strictly as a *guide* to prevent over- and under-extension of the ramp rail. They are NOT SAFETY LOCKING MECHANISMS. Simply slide the rails *slowly* in/out until the catch button pops into hole. Ramp should be used only when catch button is in or between the two holes.
- PREPARING YOUR RAMP FOR USE:** While holding single ramp rail with the top section up, push in catch button in upper hole to release top section and pull top section (see diagram) upward until catch button pops into lower hole in top section. Then, hold onto middle section and pull upward until catch button pops into hole. Check to ensure all buttons are seated inside their appropriate holes.
- Space ramp so that the tires of the wheelchair are in the center of the black non-skid tread. To make this easier, we have included a nylon cord approximately 36" long. Cut the cord the same length as the distance between the wheels of your wheelchair. The cut cord can now be used to space the ramp, both top and bottom (see DIAGRAM ITEM 2).



10. WARNING: Ensure that the turned down tab (top plate) is secure before using. We recommend attaching the ramp temporarily to the landing using the provided clevis pins (drill 5/16" hole in landing) or semi-permanently using screws through the pre-drilled holes at the top of each section (see DIAGRAM ITEM 2). To prevent ramp from becoming dislodged during use, make sure that the top plate overlaps the resting surface as far as possible. The surface holding the top plate needs to be substantial; make sure that the surface and supporting structure is of sound construction.

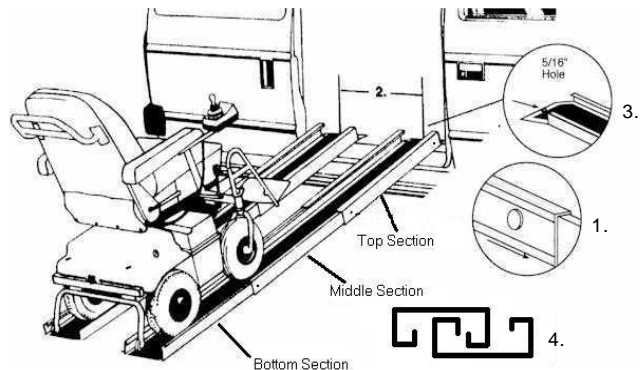

- ✂
- EZ-ACCESS, a division of Homecare Products, Inc., herein referred to as Manufacturer, warrants that the equipment is free from defects in material and workmanship under normal use and service. This limited warranty is effective for a period of 3-years from the date of original retail purchase (**original dated purchase receipt** is required) and applies to metal structure only (excludes powder coated finish and applied non-skid tread). The provisions of this warranty shall not apply to any equipment which has been subject to misuse, vandalism, negligence, alteration, accident, or improper installation, or which has been repaired outside Manufacturer's place of business in any way as, in the reasonable judgment of Manufacturer, to adversely affect its performance and reliability, nor to normal deterioration due to wear, tear and exposure, corrosion, damage caused by rain, fire, earthquake or other natural causes or acts of nature.
 - Manufacturer's obligation under this warranty is limited to repairing or replacing, at its option, any equipment which is returned to Manufacturer's place of business with return authorization number (SEE #4 FOR DETAILS) and which, upon examination, shall disclose to Manufacturer reasonable satisfaction to have been thus defective. Manufacturer will make the repair or replacement of defective parts under this warranty without charge for parts or labor.
 - If Manufacturer finds that any equipment has not provided the warranted service life, Manufacturer will, as its sole obligation and the user's sole remedy for breach of this warranty, repair or replace the unit, at its option, for a charge prorated on the following basis. The user will be allowed a credit against the list price of equivalent equipment at the time of return to Manufacturer, in proportion to the percentage of the warranted service life remaining at the time of return to Manufacturer. Purchaser will be responsible to pay the balance of the list price of equivalent equipment; and Manufacturer reserves the right to require payment prior to delivery of the repaired/replacement equipment. Manufacturer's whole warranty responsibilities are summarized below (return **shipping/delivery charges** to be paid by the Purchaser):
 - YEARS 0-1: unit repaired or replaced free of charge.
 - YEARS 1-2: 50% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
 - YEARS 2-3: 25% \$ credit (current list price multiplied by credit percentage. In calculating the available credit, the remaining warranted service life will be rounded up or down to the nearest whole year).
 - The attached registration card must be mailed within 10-days after purchase to the Manufacturer in order to validate the warranty. If Purchaser discovers a defect, he/she must call Manufacturer at 253-249-1101 and request an RA number to return, at his/her own expense, the merchandise to Manufacturer. Additionally, digital images of merchandise (close up of defect, plus photo of overall product) must be submitted (an email address will be provided at the time an authorization number is given). The duration of any implied warranty of merchantability or fitness for a particular purpose shall be limited in all respects to the duration of the limited warranty and the warranty described above shall be in lieu of any other express warranty. Some states do not allow limitation on how long an implied warranty will last so the above limitation may not apply to you. Manufacturer neither assumes nor authorizes any other person to assume for it any other liability in connection with Manufacturer.
 - The buyer's sole and exclusive remedy against the Manufacturer shall be for the repair or replacement of defective merchandise as provided above. No other remedy, including but not limited to, incidental or consequential damages for lost profits, lost sales, injury to property, or any other incidental or consequential loss shall be available to him/her. Because some states do not allow the exclusion or limitation of incidental or consequential damages, the above limitation or exclusion may not apply to you. Before using, user shall determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.
 - This warranty gives you specific legal rights. You may also have other rights, which vary from state-to-state.

