


Synergy Simplicity

Installation and Care

Instructions

ACN# 088 609 661

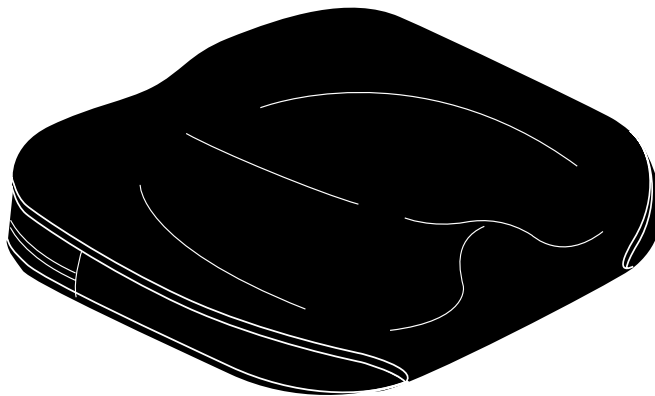

QUANTUM[®]
REHAB
INNOVATIVE REHAB SOLUTIONS

A Division of Pride Mobility Products[®] Corp.

INFORMATION LABELING

The symbols below are used throughout this owner's manual and on the power chair to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING! Failure to follow designated procedures can cause either personal injury, component damage, or malfunction (black symbol on yellow triangle with black border).



Read and follow the information in the owner's manual.

Copyright © 2004
Pride Mobility Products Corporation
INFMANU2872/Rev A/Oct 04

TABLE OF CONTENTS

INTRODUCTION 4

THE SYNERGY SIMPLICITY 6

CLEANING 7

STORAGE..... 8

WARRANTY 8

INTRODUCTION

WELCOME to Quantum Rehab, a division of Pride Mobility Products Corporation (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** use and care for this product should bring you years of trouble free operations and service.

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to use this product for the first time. In addition, your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

This manual is to be used in addition to the owner's manual that came with your chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your Quantum Rehab Specialist. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new Pride product, and about the service you received from your Quantum Rehab Specialist. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your Pride product.

Please feel free to contact us at the address below:

Pride Mobility Products Corporation
Attn: Customer Care Department
182 Susquehanna Avenue
Exeter, PA 18643-2694

NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.

My Quantum Rehab Specialist Is:

Name: _____

Address: _____

Phone Number: _____

Purchase Date: _____

THE SYNERGY SIMPLICITY

The Synergy Simplicity is labeled to assure proper placement on the chair.

To place the Synergy Simplicity on the chair:

1. Locate the “Back Bottom” label on the underside of the cushion.
2. Place the cushion on the seat so that the “Back Bottom” label is located on the back of the seat, label side facing down. The zipper on the cushion will face the rear of the seat. See figure 1.

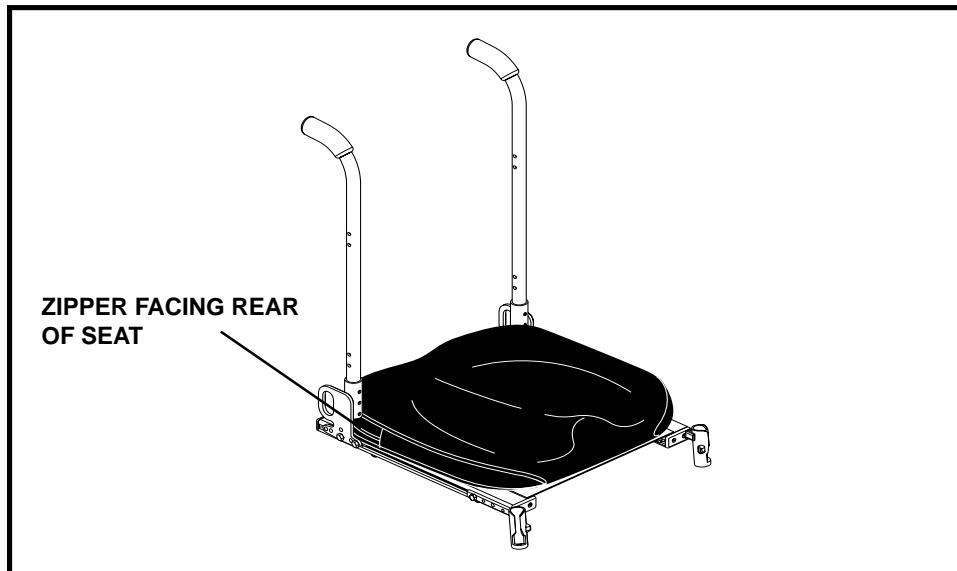


Figure 1. Cushion Placement

CLEANING

The Synergy Simplicity can be easily cleaned.

To clean the Synergy Simplicity:

1. Locate the zipper on the rear of the cover. See figure 2.
2. Unzip the cover and pull out the inner cushion.
3. Wash the cover in a washing machine on the COLD setting.



WARNING! Do not use bleach when washing the cover. Doing so may cause damage to the cover.

WARNING! Never submerge the inner cushion of your Synergy Simplicity in water. Doing so may cause damage to the cushion.

4. Allow the cover to air dry.



WARNING! Do not attempt to put the cover in a dryer. This may cause shrinking.

5. Once dry, put the inner cushion back inside the cover. Zip the cover closed, and place the Synergy Simplicity back onto the seat of your chair.

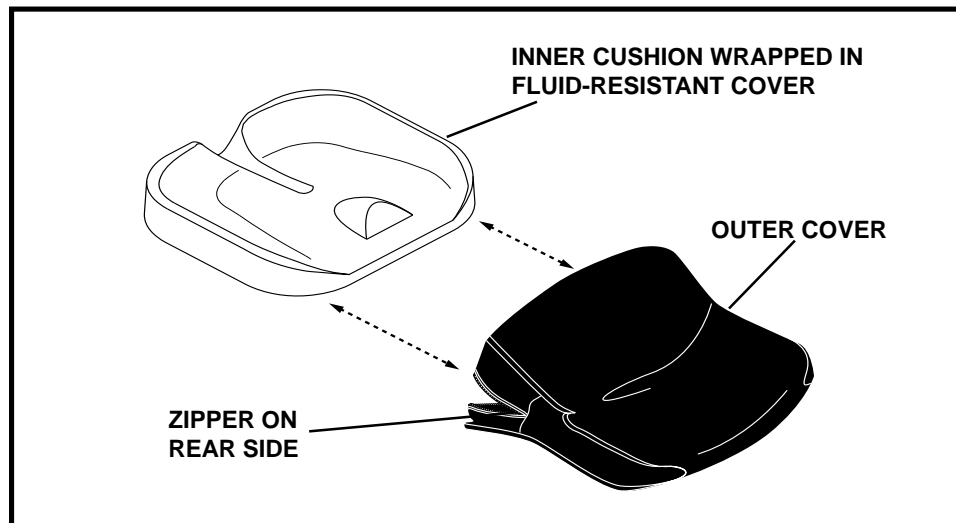


Figure 2. Cover Removal

STORAGE

The Synergy Simplicity should not be stored in extreme heat or cold temperatures. It should be stored at room temperature in a clean and dry location.

WARRANTY

The Synergy Simplicity cushion is designed and produced to the highest of standards. Our Synergy Simplicity inner cushion is guaranteed to be free of defects in material and workmanship for a period of 24 months. The outer cover is guaranteed to be free of defects in material and workmanship for a period of 90 days. If any defect in material or workmanship is found, Pride Mobility Products will repair or replace, at our discretion, the Synergy Simplicity cushion. This warranty does not apply to cushions abused or misused by the user and deemed such by Pride Mobility Products.

Claims and repairs should be processed through the nearest authorized Pride Provider.

NOTES

NOTES

NOTES

QUANTUM[®]
REHAB
INNOVATIVE REHAB SOLUTIONS
A Division of Pride Mobility Products[®] Corp.

www.quantumrehab.com
www.pridemobility.com

Pride Mobility Products Corporation
182 Susquehanna Avenue
Exeter, PA 18643
USA

Pride Mobility Products Company
380 Vansickle Road Unit 350
St. Catharines, Ontario L2R 6P7
Canada

Pride Mobility Products Ltd.
Unit 106, Heyford Park Camp Road
Upper Heyford, Oxfordshire OX25 5HA

Pride Mobility Products Australia Pty. Ltd.
21 Healey Road
Dandenong, 3175
Victoria, Australia

Pride Mobility Products Italia s.r.l.
Via della Tecnica, 14
Prato della Corte
00065 Fiano Romano
Roma - Italia

Pride Mobility Products Europe B.V.
Tijmuiden 28
1046 AL Amsterdam
The Netherlands

Pride